


## Stryker Evacuation Stair Chair Instructions

These instructions are printed on the backrest of the Evacuation Chair.

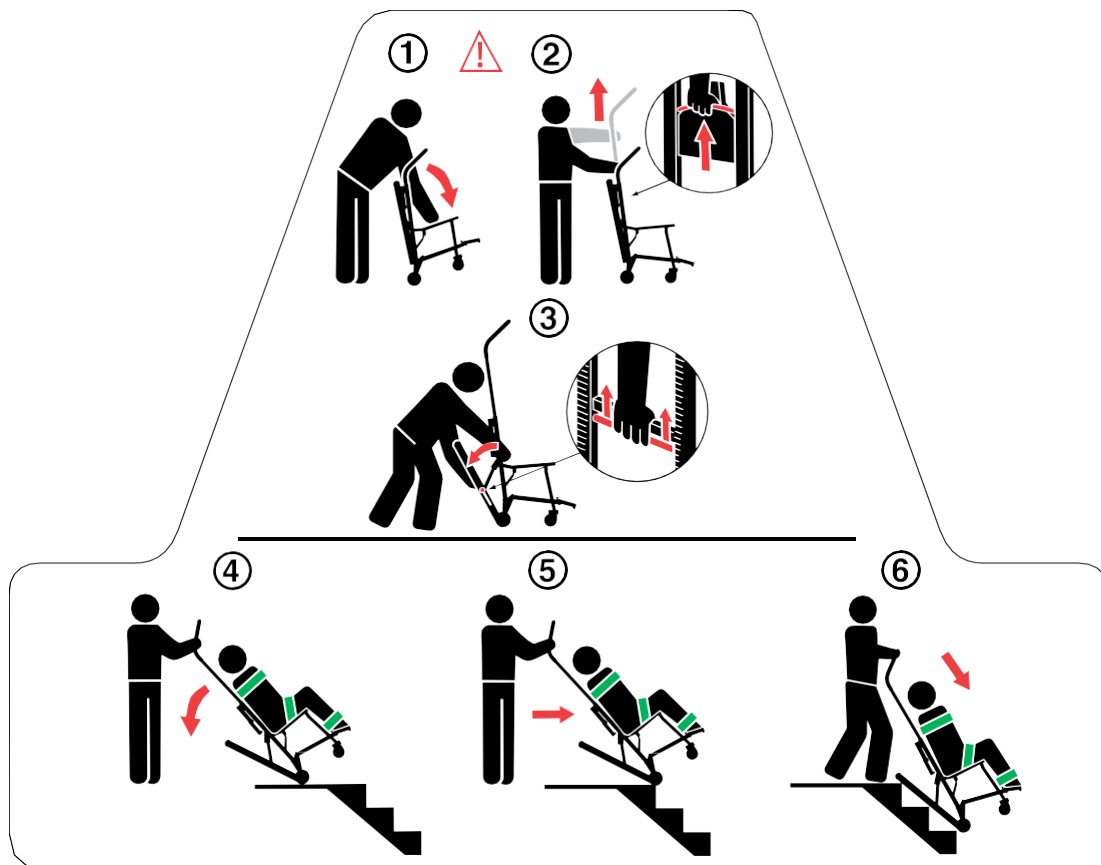

### To use the evacuation chair:

1. Stand behind the chair and pull the backrest and extension handle apart to unfold the chair.

***Warning*** An unlocked chair can fold during use, causing injury to the passenger or operator. Always make sure the chair is locked in the unfolded position before use.

2. Pull the red release cable with one hand and pull up on the control handle with the other hand to extend the upper control handle.
3. Squeeze the red track release bar against the black cross tube. Relax your grip on the release bar and forcefully pull the Stair-TREAD™ system to the fully extended position until both sides lock securely. Always verify both sides of the Stair-TREAD system are locked by trying to fold it back up.

***Caution*** Release the red track release bar before clicking the Stair-TREAD system into the locked position. Failure to follow this procedure could result in the track failing to

*lock. Always verify the Stair-TREAD system is locked by trying to fold it before descending stairs.*

4. Tilt the chair back just far enough to allow the Stair-TREAD system to contact the floor.
5. Maintaining the angle, guide the Evacuation Chair over the edge of the stairs, allowing the Stair-TREAD system to engage the first step.
6. Glide down the stairs until the treads are level across the edges of two or three steps.